**BHL-Singleflow®**

Extensive line with check valve and LLMM - REF: BHL-SVM-20, REF: BHL-SVM-30, REF: BHL-SVM-60, REF: BHL-SVM-120, REF: BHL-SVM-150, REF: BHL-SVM-200, REF: BHL-SVM-250

Extensive line with double check valve and LLMM - REF: BHL-DVM-20, REF: BHL-DVM-30, REF: BHL-DVM-60, REF: BHL-DVM-120, REF: BHL-DVM-150, REF: BHL-DVM-200, REF: BHL-DVM-250

Extensive spiral line with 1 check valve - REF: BHL-SVC-150, REF: BHL-SVC-200

Extensive spiral line with 2 check valve - REF: BHL-DVC-150, REF: BHL-DVC-200

**Indications**


Patient lines *Singleflow®* are destined for Radiology procedures, CT and MRI, with administration of Contrast Media. Patient lines *Singleflow®* have security valves that assure a barrier against contamination between the patient and the administration device. They are single use devices for patient. Devices must be used only by personnel trained for the administration of contrast media. Medical personnel are responsible for cross contamination prevention and accomplishment of with aseptic procedures to reduce the risk of contamination. Informations supplied with the medicines to be administered must be taken in consideration before any procedure execution. The devices are Luer connections. The safety use of the patient lines *Singleflow®* is assured by the following protocol.

**Protocol**

1. Open the package and remove the protection cap from the female connector side of the patient line *Singleflow®*;
2. Connect the female connector to the terminal part of the administration device to use;
3. Purge the device and assure all the air is removed from the tubing;
4. Connect the male connector of the patient line *Singleflow®* to the needle or to the catheter to use on the patient;
6. Assure that all the connections are correctly executed and start the radiologic procedure;
7. When the procedure is finished, disconnect the patient line *Singleflow®* used;
8. Place the patient line *Singleflow®* used into the hospital residuals recipient.

**Warnings!**

- Do not exceed the indicated working pressure of 300Psi. The use of a higher pressure, or occlusions may result in leaks or device breach.
- Needles or catheters with reduced flow rate affect the working pressures.
- It's recommended the use of patient lines *Singleflow®* with administration devices *Multiflow®*.
- Before start a new procedure check if the patient lines was changed;
- After use the devices must be treated as hospital waste.
- The manufacturer is not responsible for damages that may arise from the device misuse or for the non accomplishment of the Instructions for use.
- Defined storage conditions must be kept and assured.

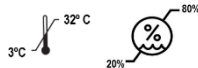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