Estrada da Avessada n.º 390  
2665-402 Santo Estêvão das Galés, Portugal  
Phone: +351 219821694  
web: www.bhl-medical.com

**Perfusion System with aerator - P18D - REF: 1100117A**  
**Perfusion System with aerator - P18D NSK - REF: 1100117C**  
**Perfusion System with aerator - P18D WH - REF: 1100117D**

#### Indications

Perfusion systems with aerator, with commercial designation P18D, are intended for washing and irrigation when connected to an infusion pump commonly known as "Implant motor".

These products were designed to support dental techniques, allowing the washing and irrigation of an area under treatment, benefit the blood removal, tissue fragments and clean wounds after the intervention.

They are composed by a spike that enable an easy penetration of container ports, tube to allow the fluid to flow through, roller clamp to control the flow of liquid, and an integrated part made of flexible tube that allows the connection to the pump.

The end part is constituted by a thin tube that can easily be adapted to the hand piece of the pump.

**References: 1100117A and 1100117D are compatible with Wellton & Healthcare (W&H) implant motors.**

**Reference: 1100117C is compatible with Nakanishi (NSK) implant motors.**

The safety use of the Perfusion systems with aerator is assured if the protocol described below is followed.  
Before start the protocol described, the Instructions of use related to the infusion pump must be taken in consideration.

#### Protocol

1. Open the pump cover. Insert the flexible tube, with pump connectors joint, into the pump.
2. Close the pump cover.
3. Connect the spike of the drip chamber to the bottle/bag with the solution for irrigation.
4. Connect the thin tube, in the end part of the perfusion system, to the hand piece of the pump.
5. Purge the system and ensure the air is removed from the tubing system.
6. Start the procedure.
7. When the procedure is finished disconnect perfusion system from the pump.

#### Warnings!

- Perfusion systems with aerator *must be used* only by trained personnel. Medical personnel are responsible for preventing any cross contamination. Asepsis procedures must be strictly complied to reduce the risk of contamination.
- These products were not designed for medicines administration. They must be used only for serum or sterile water.
- These products are completely phthalate free.

Use by date

Batch code

Catalogue number

Sterilized using ethylene oxide

Single use

Do not use if package is damaged

Keep away from sunlight

Keep dry

Consult instructions for use

Caution

Estrada da Avessada n.º 390  
2665-402 Santo Estêvão das Galés, Portugal  
Phone: +351 219821694  
web: www.bhl-medical.com

**Perfusion System with aerator - P18D - REF: 1100117A**  
**Perfusion System with aerator - P18D NSK - REF: 1100117C**  
**Perfusion System with aerator - P18D WH - REF: 1100117D**

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