Extensive Lines

Device designation

- Polyvalent extensive line with male/female connector
- Pediatric extensive line with male/female connector
- Arterial extensive line with male/female connector

Indications

Extensive lines are destined to be used as an extension of infusion devices for administration of fluids.

Single use device. Do not reuse. Do not re-sterilize.

Devices must be used only by trained personnel.

Medical personnel are responsible for cross contamination prevention and accomplishment of aseptic procedures to reduce the risk of contamination.

The device is a Luer connection and latex free.

The safety use of the extensive line is assured by the following protocol.

Protocol

1. Open the package and remove the protection cap from the female connector side of the extensive line;
2. Connect the female connector to the terminal part of the infusion device to use;
3. Purge the devices and assure all the air is removed from the tubing;
4. Connect the male connector of the extension line to the needle or to the catheter to use on the patient;
6. Assure that all the connections are correctly executed and start the procedure;
7. When the procedure is finished, disconnect the extension line used;
8. Place the extensive line used into the hospital residuals recipient.

Warnings!

- Device is not intended for reuse. Intended to be used once for a single patient.
- After use the devices must be treated as hospital waste.
- The manufacturer is not responsible for damages that may arise from the device misuse or for the non accomplishment of the Instructions for use.
- Defined storage conditions must be kept and assured.

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