## Extensive Lines

### Device designation

- Polyvalent extensive line with male/male connector
- Pediatric extensive line with male/male connector
- Arterial extensive line with male/male connector

### Indications

Extensive lines are destined to be used as an extension of infusion devices for administration of fluids.

Single use device. Do not reuse. Do not re-sterilize.

Devices must be used only by trained personnel.

Medical personnel are responsible for cross contamination prevention and accomplishment of aseptic procedures to reduce the risk of contamination.

The device is a Luer connection and latex free.


The safety use of the extensive line is assured by the following protocol.

### Protocol

1. Open the package and remove the protection cap from the male connector side of the extensive line;
2. Connect the male connector to the terminal part of the infusion device to use;
3. Purge the devices and assure all the air is removed from the tubing;
4. Connect the male connector of the extension line to the needle or to the catheter to use on the patient;
6. Assure that all the connections are correctly executed and start the procedure;
7. When the procedure is finished, disconnect the extension line used;
8. Place the extensive line used into the hospital residuals recipient.

### Warnings!

- Device is not intended for reuse. Intended to be used once for a single patient.
- After use the devices must be treated as hospital waste.
- The manufacturer is not responsible for damages that may arise from the device misuse or for the non accomplishment of the Instructions for use.
- Defined storage conditions must be kept and assured.

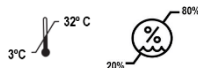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